

ECONOMICAL

HANDS-FREE

FAST

EASY TO USE

QUIET

MULTI-FUNCTIONAL



Introducing...

A Revolution in Pill Counting.



C-Ray

Electronic Pill Counter



N1671 Powers Lake Road, Powers Lake, WI 53159
P 1.262.279.5307 • F 1.262.279.5360 • email: mail@tcgrx.com
www.tcgrx.com

TCG is proud to present the *C-Ray Electronic Pill Counter*.



1. Pour Stock into Counter

This revolutionary, modern design enables pharmacists to greatly reduce the time required to fill prescriptions or count inventory. Thus, allowing pharmacists more time to tend to the needs of clients. Small and economical enough to place at each dispensing station.

TCG's Technological Advantages:

- Economical
- Hands-free Counting
- Counts Scripts
- Rapid Counting (*100 pills in less than 25 sec.*)
- No Adjustments Required
- Counts Inventory
- Quiet



2. Pills Count Automatically



3. Return Stock



4. Fill Vial

Easy to Clean

All pill pathways are visible with easy access for cleaning.

To clean simply remove:

- Top cover
- Spiral
- Bowl
- Both pill trays

Once all the above components have been removed, they can be individually cleaned using a lint free cloth dampened with isopropyl alcohol.



Simple to Use

1. Enter prescribed amount.
2. Select pill type.
3. Press *Start/Stop* button.
4. Pill count is displayed in *Count* window.
5. Status is displayed in the top four indicators.
6. Count may be paused by pressing the *Pause/Cancel* button.
7. Counting is automatically paused when more pills are required.
8. Count can be aborted by pressing the *Start/Stop* button.



C-Ray Specifications:

Size	9.5" width x 15.7" depth x 9" height
Weight	3 Kg (6.6 Lbs)
Power Requirements	12V DC, 1.2A
Certifications:	CSA, UL, CE, FCC